Bedienungsanleitung / Manual

HYGOMAT

- ▶ CLASSIC
- ▶ COMFORT
- ▶ CLEANROOM

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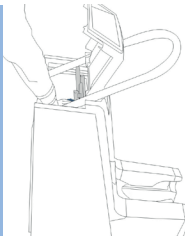
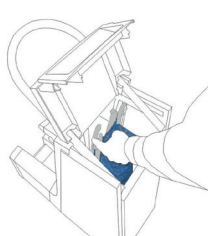
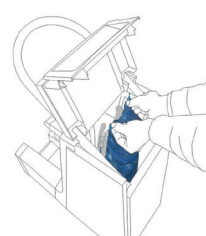
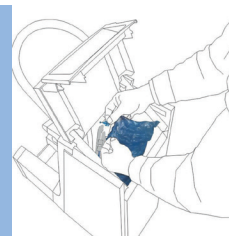
I-12

INSTALLATION

1. Assemble the machine as shown in image I-1. Open the cover as shown in image I-2.
2. For Comfort/Cleanroom: attach the hand rail with the screws, shown in image I-2. The guidance rails as shown in image I-2 can be manually adjusted with a Phillips screwdriver/Allen key. **Important notice for overshoes 28502H/04H:** Loosen the screw manually. Slide the illustrated insertion rails approx. one centimetre outwards so that the overshoes fit ideally into the rails (image I-2A) and fix the screw again.
3. Now take one bundle of overshoes with both hands and slide the plastic holdings into the guidance rail on both sides. Shown in image I-3. (In case the gap is not correct, it can be adjusted with the left and right sides of the plastic holders.)
4. Press the bundle of overshoes slightly forward and then hold it straightly to make sure that all plastic holders are located inside of the guidance rail. Shown in image I-4.
5. Steadily press down the bundle on both sides and with both hands into the guidance rail until all overshoes are arranged under the locking spot and inside of the guidance, as shown in image I-5.

6. Untie the ribbon holding the overshoes together and remove it as shown in diagram I-6.
7. Remove the cardboard as shown in diagram I-7.
8. Remove the plastic locking strip on both sides of the overshoes in place with one hand. Use the other hand to grip the plastic ring and pull it completely out of the machine as shown in diagram I-8.
9. Push both sides of the bundle with both hands and press all the overshoes inside the tracks until the last piece is completely inserted into the track, over the lock position, as shown in diagram I-9.

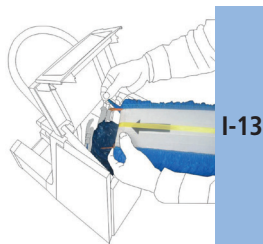
HYGOMAT COMFORT/CLEANROOM – OPTIONS TO PROCEED:

10. The first shoe cover of the second bundle has to be attached to with the last shoe cover of the first bundle.
11. Please guide the last shoe cover along the guide rail as shown in image I-9.
12. Open the locks pushing them downwards.
13. Take the two plastic holders out of the last shoe cover of the bundle as shown in image I-11.

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
14. Remove the 2 plastic holders of this shoe cover as shown in diagram I-12.
15. Remove the plastic holders from the second bundle from the right plastic holder side as shown in diagram I-13.
16. Attach the first plastic holder of the second bundle to the last shoe cover of the first bundle as shown in diagram I-14.
17. Guide the plastic holder of the second bundle along the guide rail and push it enough so that far as the lock is in the guide rail as shown in diagram I-15.

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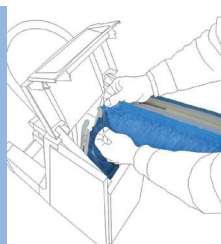
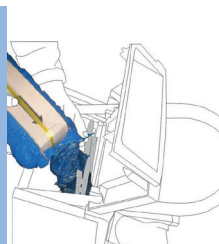
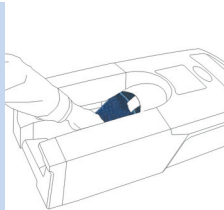
PRAY CONTINUE:

18. Repeat Step 16 to 18 for the left side of the plastic holder stripe.
19. Repeat step 2 to 7 for the second shoe cover bundle.
20. Close the coverage and pull the first shoe cover out of the opening, which is underneath the metal plate as shown in diagram I-16.
21. Guide this shoe cover along the guiding rail until it reaches the hanger as shown in diagram I-17.
22. Now the overshoe-dispenser is ready as shown in diagram I-18.

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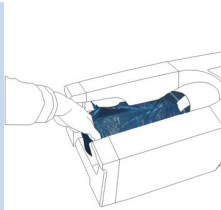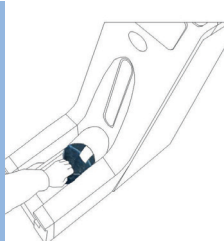
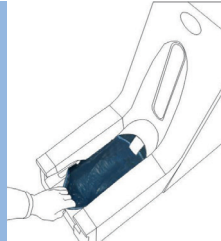
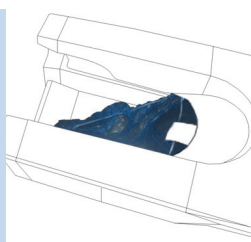
MOST COMMON ISSUES:

PROBLEM: One or boths sides of the plastic holders are not in the guiding rail as shown in diagram I-19.

POSSIBLE REASON: The plastic holders have not exactly been fixed into the guiding.

SOLUTION: Please make sure that all plastic holders are in the guiding rail as shown in diagram I - 20. If not, please push down both locks, which are fixed on the guiding rail, to loose them. Remove the loose overshoes and fix them again put them in again.

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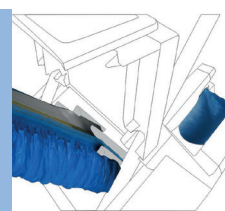
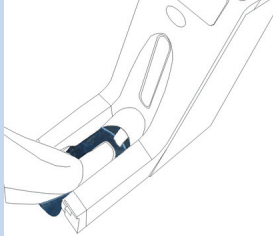
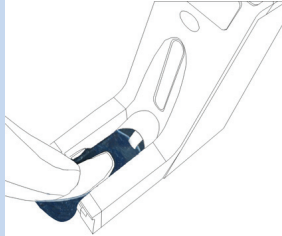
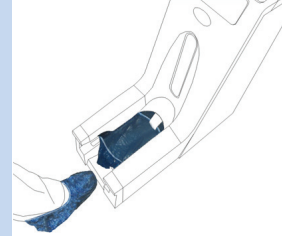
IMPORTANT NOTICE:

1. During the installation of overshoes, please make sure that the plastic clips are all inside the tracks.
2. During use, put the foot inside the shoe cover and pull the shoe cover backwards in horizontal direction. Do not lift the front of the foot too high.

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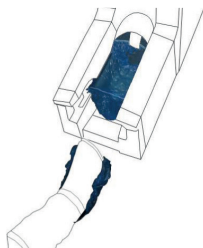
PROBLEM SOLUTION

PROBLEM: The shoe is only half inside the shoe cover as shown in diagram I-24.


POSSIBLE REASONS: When pulling the shoe cover backwards, the user lifted the front of the foot too high, which is why the foot is not inside the shoe cover, as shown in diagram I-25.

SOLUTION: Put the foot inside the shoe cover and pull the shoe cover backwards in horizontal direction as shown in diagram I-26.


HYGOMAT CLASSIC/
HYGOMAT COMFORT/
CLEANROOM

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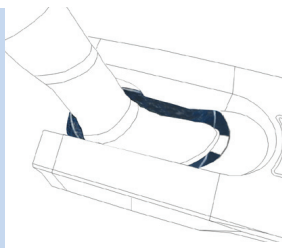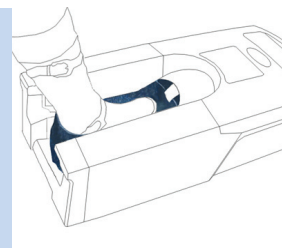ONLY FOR HYGOMAT CLEANROOM:

Hygomat Cleanroom can be cleaned with all common detergents and be disinfected due to its stainless steel side parts and its stainless steel backside.

ASSEMBLING THE ROLLS:

For assembling the rolls please use all 4 screws.